

BEPPI

EN

Bepi condoms are made of high quality natural rubber latex, smooth surface, natural colour, reservoir ended and lubricated with non-medicated lubricant. When used in accordance with manufacturer's instruction, condoms are an effective form of contraception. However, no method of contraception can provide 100% protection. If used properly, latex condoms will help to reduce the risk of transmission of HIV infection (AIDS) and many other sexually transmitted diseases, including chlamydia infection, genital herpes, genital warts, gonorrhoea, hepatitis B and syphilis. *Nominal width: 56 ± 2 mm.*

Precaution and Warnings

- The condom is for single use only.
- Do not use oil-based lubricants, such as petroleum jelly, massage oil, baby oil, body lotion, butter, etc. which can damage condoms. So can some medicines that are applied to penis or vagina. Consult your doctor or pharmacist if it is safe to use these medicines with condoms.
- If experience irritations, discontinue use of the latex condoms and get advice from your doctor.
- Seek medical assistance, at least within 72h, should a condom leak or burst during use.
- If the individual foil is obviously damaged, discard the condom.
- Store in a cool, dry place away from heat and direct sunlight.
- Stop and check if the condom is slipping off or tighten excessively onto the penis, because this may lead to breakage.

Instruction for Use

STEP 1 Check the expiry date before usage. Tear along one side of the foil, be sure not to rip the condom. Carefully handle the condom as it can be damaged by fingernails, jewellery etc.

STEP 2 Put the condom on after the penis is fully erect, before there is any contact between the penis and your partners body. This helps to prevent pregnancy and transmission of STIs. Air trapped inside a condom, could cause it to break. To avoid this, squeeze the tip of the condom. Be sure that the roll is on the outside and place the condom over the erect penis. While still squeezing the tip, use your other hand to unroll the condom gently pressing air out towards the base of the penis.

STEP 3 Soon after ejaculation, while the penis is still erect, hold the condom firmly in place at the base of the penis as you withdraw, so that the condom does not slip.

STEP 4 Wrap the used condom in a tissue and dispose into a waste bin. Do not flush it down the toilet.

Caution: This condom is made of natural rubber latex which may cause allergic reactions including anaphylactic shock. If user is allergic for latex.



keep dry



keep away
from sunlight




do not
reuse



Contains
natural rubber
latex



do not use if
package is
damaged

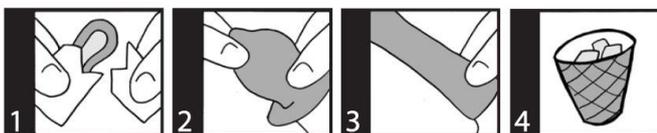

ASHA International
Seinhuiswachter 1-3, 3034 KH
Rotterdam, The Netherlands

Standard EN ISO 4074:2015
100% Electronically Tested
Nominal Width: 56 ± 2mm

8-7-2019